

# Pluslife Influenza A/Influenza B/RSV Card One nucleic acid test card


One nucleic acid test card Three Respiratory Viruses Identification

Influenza A

Influenza B

Respiratory syncytial Virus



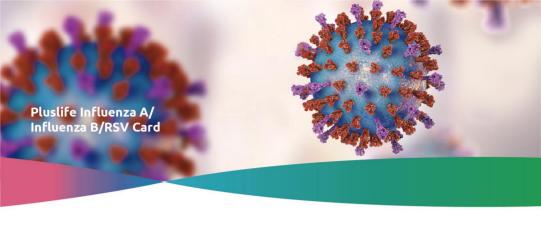

Pluslife Mini Dock is a rapid molecular diagnostic system that can be used to detect Influenza virus A(IVA), Influenza virus B(IVB) and Respiratory syncytial virus(RSV) in one single test. Our self-developed isothermal amplification technology, RHAM, provides excellent results comparable to qPCR tests. The product is compact and portable, with easy-to-operate procedure to obtain results within 35 minutes.

#### **Features**




## **Test Procedure**



#### 1. Collect Sample

Collect the sample with nasal swabs, nasopharyngeal swabs or oropharyngeal swabs

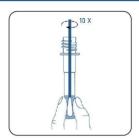

#### 2. Mix Sample

Insert the swab into sample tube, mix and squeeze 10 times, and dispose



#### 3. Prepare the Reaction Card

Transfer the sample to the chambers of the reaction card



#### 4. Run Test

Turn on the device and insert the reaction card to start the test

## **Result Output**



✓ Mini Dock



✓ PC Software



✓ Mobile App

# **Application**









Test Centers

Others

# **Specification**

Pluslife Mini Dock





T tustife Milli Dock	
Dimension	101 x 91 x 65 mm
Weight	210 g
Connection	Computer software or mobile app
Operate temperature	15°C-30°C
Operate humidity	≤ 80%

T tostile lilitaeriza Ayillitaeriza D/RSV Cara	
Target pathogen	IVA, IVB, RSV
Limit of detection (LoD)	1000 copies/mL
Turnaround time (TAT)	35 mins
Shelflife	9 months at room temperature
Sample type	Nasal / Nasopharyngeal / Oropharyngeal swab

Pluslife Influenza A/Influenza B/RSV Card

## Guangzhou Pluslife Biotech Co.,Ltd.

















