


# Pluslife Strep A Card

**Group A Streptococcus** 

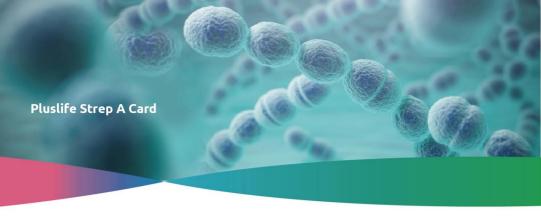



Pluslife Mini Dock is a rapid molecular diagnostic system that can be used to detect Group A Streptococcus (GAS). Our self-developed isothermal amplification technology, RHAM, provides excellent results comparable to qPCR tests. The product is compact and portable, with easy-to-operate procedure to obtain results within 35 minutes.

#### **Features**



### **Test Procedure**



#### 1. Collect Sample

Collect sample with oropharyngeal swabs



#### 2. Mix Sample

Insert the swab into sample tube, mix and squeeze 10 times, and dispose



### 3. Prepare the Reaction Card

Transfer the sample to the chambers of the reaction card



#### 4. Run Test

Turn on the device and insert the reaction card to start the test

### **Result Output**



✓ Mini Dock



✓ PC Software



✓ Mobile App

# **Application**







Others

# **Specification**



Pluslife Mini Dock



1 tustil C Milli Dock	
Dimension	101 x 91 x 65 mm
Weight	210 g
Connection	Computer software or mobile app
Operate temperature	15°C-30°C
Operate humidity	≤ 80%
	7.0

Pluslife Strep A Card	
Target pathogen	Strep A
Limit of detection (LoD)	1000 copies/mL
Turnaround time (TAT)	35 mins
Shelf life	13 months at room temperature
Sample type	Oropharyngeal swab

### Guangzhou Pluslife Biotech Co.,Ltd.







+86-020-31703986 market@pluslife.com www.pluslife.com









PLGAS-INT-20230404